# Instructions For Use

## Specs IC

Issue Date: May 2, 2025

### Revision Table:

| Revision | Date       | Description |
|----------|------------|-------------|
| 1.0      | 05/02/2025 | Issue Date  |

Model: 10000114

UDID: 850011150382

|                                                                                                                                                                                                                                             |                                                                                                                                                                                       |    |     |                                                                                                                                                  |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----|-----|--------------------------------------------------------------------------------------------------------------------------------------------------|
| <br><br>AbleNet, Inc.<br>2625 Patton Road<br>Roseville, MN 55113<br>United States of America<br><a href="http://www.ablenetinc.com">www.ablenetinc.com</a> | <table border="1" data-bbox="712 905 904 966"><tr><td>EC</td><td>REP</td></tr></table><br>EUCEREP<br>Roald Dahllaan 33, 5629MC<br>Eindhoven<br>The Netherlands<br>eucerep@eucerep.com | EC | REP | <b>UK Responsible Person</b><br><br>EUCEREP LTD<br>7 Tibbets Close, Meeting Lane<br>Alcester, Warwickshire<br>B49 5QU<br>England, United Kingdom |
| EC                                                                                                                                                                                                                                          | REP                                                                                                                                                                                   |    |     |                                                                                                                                                  |

|    |     |
|----|-----|
| CH | REP |
|----|-----|

MedEnvoy Switzerland  
Gotthardstrasse 28  
6302 Zug  
Switzerland  
Telephone: +41 41 562 01 42

## INTENDED PURPOSE

This switch is used to give access to items that are not normally accessible because of issues with cognitive or motor control.

## CLINICAL BENEFITS OF THE DEVICE

Allows patients to access technology when they would not have the motor or cognitive skills to operate another device.

## PERFORMANCE CHARACTERISTICS AND INFORMATION FOR HEALTH CARE PROFESSIONAL

**Switch Type:** Mechanical : Momentary

**Connection Type:** Wired

**Cord Length:** 6 ft/183 cm

**Activation Type:** Pressure

**Activation Surface:** 1.375-in/3.5-cm diameter

**Activating Force:** 3.92-oz/111-g

**Travel Before Activation:** 0.029-in/0.74 mm

**Feedback:** Auditory click, Tactile

**Plug Size:** 1/8-in/3.5-mm mono

**Circuit:** Normally Open

**Mounting Connection:** Quick Ready Mounting Plate for Assistive Technology

**Product Dimensions:** 1.94-in L x 1.94-in W x 0.6-in H / 5-cm L x 5-cm W x 1.6-cm H

**Material:** Housing - ABS Plastic

**Suggested Operating Environment:**

Ambient Temperature Range: 32°F - 95°F (0°C - 35°C)

Humidity: 10% - 95% (no condensation on device)

**Suggested Transportation and Storage:**

Ambient Temperature Range: -25°F - 158°F (-31°C - 70°C)

Humidity: 10% - 95% (no condensation on the device)

## RISKS, WARNINGS, AND CONTRA-INDICATIONS

Specs IC Switch is neither waterproof nor completely water resistant. Do not immerse Specs IC Switch in water or any other liquid. Make sure that no liquids are spilled on Specs IC Switch.

Specs IC Switch is not recommended to be used as access to a call system.

Specs IC Switch should only be plugged into low voltage, low current applications. Do not plug switch into a jack that is high voltage.

Do not plug switch directly to mains power.

DEVICE OVERVIEW


|   |                                |
|---|--------------------------------|
| 1 | Switch Top                     |
| 2 | 1/8-in/3.5-mm mono switch plug |

|   |             |
|---|-------------|
| 3 | Release Tab |
| 4 | White Dot   |
| 5 | Spring      |

|   |             |
|---|-------------|
| 6 | Base 1      |
| 7 | Base 2      |
| 8 | Strap       |
| 9 | Screwdriver |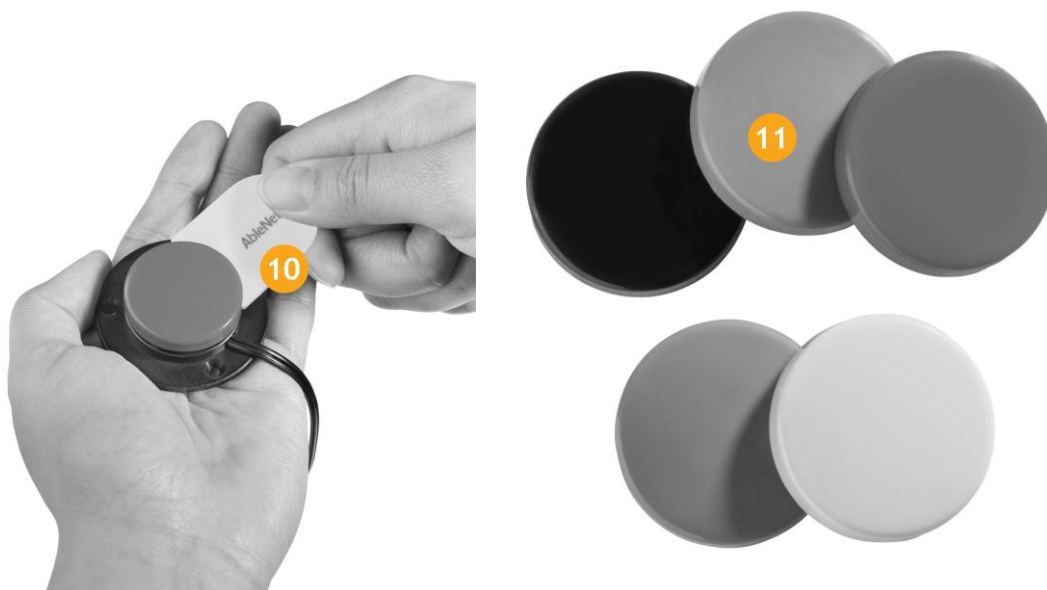

|    |              |
|----|--------------|
| 10 | Release Tool |
| 11 | Switch Caps  |

## USE INSTRUCTIONS

## First Time Use

1. Place a colored Switch Cap on top of the spring.
2. Gently push down on the switch cap and turn it clockwise until you hear it click. The switch cap will not be able to spin in either direction once it is correctly locked in place.
3. Plug Specs IC switch into any switch adapted device with a standard, mono 3.5mm switch jack.
4. Once the activation area is pressed, any attached device will activate.

## Changing the Switch Top

1. Locate the white dot on the switch base. This is the location of the Release Tab.
2. Using the Release Tool, press firmly on the Release Tab while turning the Switch Cap counterclockwise.

## Mounting

The Specs IC switch can be mounted in a variety of different ways using the 3 interchangeable bases and 24-inch velcro strap. For flat surfaces and simple mounting, try Velcro® or Dual Lock.

The Specs IC comes with a base attached. To remove the base, remove the screws using the provided screwdriver.

To use the velcro strap:

1. Thread the velcro strap through the slits on the Velcro Strap Base with the loop (or fuzzy) part of the velcro facing the underside of the base.
2. Turn the Specs Switch over and remove the 2 screws from the attached large base.
3. Place the strap base onto the switch by aligning the small bump on the base with the small hole on the switch.
4. Replace screws.

## PREPARATORY TREATMENT

Cleaning and disinfecting your AbleNet assistive technology devices between uses is essential to the health and safety of the user. To help fight against the spread of germs, here are some helpful steps to cleaning and disinfecting your AbleNet device.

Always be sure to wear the appropriate personal protective equipment such as protective gloves and safety glasses.

## Cleaning Your Device

Before disinfecting your device, remove any dirt or debris that may be on the device.

1. Wipe down the device with cleaning wipes.
2. Use a cotton swab soaked with cleaner to get into any tight areas that were not accessible with the cleaning wipes.
3. Let the device completely dry.

## Disinfect Your Device

Now that the device is clean, it is time to disinfect it. Changing protective gloves may be required before disinfecting the device to ensure no dirt or debris is transferred back to the device.

1. Make sure the device is still turned off and that the batteries are removed.

2. Use a clean towel or wipes with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the surfaces of the device. It is important to note that these chemicals only disinfect when they are wet and should be left wet on the device for at least 4-minutes.
3. Use a cotton swab soaked with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the tight areas that were not accessible with the towel or wipes. Make sure the chemical stays wet on the device for at least 4-minutes.
4. Let the device air-dry entirely before using it.

## **TROUBLESHOOTING**

### **Switch is Non-Responsive**

1. Inspect cord for any physical damage.
2. Plug switch into another device that is working with another switch.

## **PREVENTATIVE AND REGULAR MAINTENANCE**

Clean the Specs IC if necessary. No scheduled maintenance is required.

## **REUSABLE**

If the device is intended to be reused, it must be cleaned and disinfected between uses in accordance with standard infection control procedures. Reuse is appropriate only if the device remains in good working condition and shows no signs of material degradation.

### **Cleaning and Disinfection Prior to Reuse**

Refer to the PREPARATORY TREATMENT section, above.

### **Inspection Prior to Reuse**

Before each reuse, inspect the device for:

- Physical damage or wear
- Signs of degradation to wiring, battery compartment, or connectors
- Corrosion, cracking, or discoloration of components

### **Discontinuation of Use**

Discontinue use and dispose of the device according to local regulations if:

- Any signs of material degradation are present
- The device fails to operate reliably

Where appropriate, devices may be reused if they are reconditioned by the manufacturer or an authorized representative, in accordance with applicable safety and performance requirements.

## **DEVICES USED TOGETHER WITH**

Switch Adapted Toys

HITCH 2

Blue 2 FT

Powerlink 4

Other adapters, devices or toys that are designed with 3.5mm-mono plug for switch use.

## **PROPER DISPOSAL**

Specs IC Switch contains electronic components and should not be disposed of in your normal home or business waste collection. Specs IC Switch should be disposed of with a licensed electronics recycler.

## **WHEN TO CONTACT A HEALTHCARE PROFESSIONAL**

If you have difficulty accessing the switches or need assistance with product placement.

Change in your condition.

## **NOTICE OF SERIOUS INCIDENT**

Notice - If any serious incident has occurred while using this device, seek medical attention and then - please report to AbleNet, Inc and your competent authority of the member state in which you are in.

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